



**INFECTION
PREVENTION
PRODUCTS**

Member of CAHF and APIC

Manufacturer Recommended Usage for the IP-Pouch

IP-Pouch Application Protocol effective August 2013

1. Write-in date when the IP-Pouch is installed. Use a permanent marker include Day/Month/Year.
2. The IP-Pouch is to be secured in close proximity to the patient for easy access.
3. The IP-Pouch can be adhered to oxygen concentrators, wheelchairs and bedside tables etc. Attaches 3 ways: Peel & Stick Tape, Drawstring, or Velcro Straps.
4. To protect the patients, staff & the facility from cross contamination, always store Nasal Cannulas & HHN's inside the IP-Pouch when not in use.
5. The IP-Pouch is to be replaced once per month per patient or PRN. (Replace when visibly soiled or if airflow is impeded).

Call if you have any questions:

Infection Prevention Products Inc.

1124 Almond Vista CT.

Chico, CA 95973

www.IPPmfg.com

1-888-587-7775

